



User's manual

Manual del usuario

Manuel de l'utilisateur

Customer Service  
US: 1-800-645-2986

Servicio de atención al Cliente  
US: 1-800-645-2986

Service à la clientèle  
Canada: 888-645-2986

## 19" Transport Chair, Black

Models: 695986

### 1. List of Parts:

1. Push Handles
2. Half-Folding Back Mechanism
3. Armrest
4. Seat belt
5. Swing away footrest
6. Brake
7. 8" Rear wheel
8. 8" Front Castor
9. Footplate



### 2. Introduction

Thank you for purchasing the Global Industrial Transport chair. The Global Industrial Transport chair has been designed for occasional use both indoors and outdoors and will take a maximum user weight of 113kg (250lbs). It is a transit wheelchair and can only be operated by the wheelchair attendant/ carer.

### 3. Specification

Wheelchair Dimensions: 49cm (Width) x 84cm (Length) x 99cm (Height)  
Seat Dimensions 46cm (Width) x 40cm (Length) x 48cm (Height)  
Folded Width: 20cm  
Weight: 13.5kg  
Maximum User Weight: 113kg  
Tire type: Solid PU

# 19" Transport Chair

## 4. Assembly

This transport chair will come fully assembled with the exception of the leg rests. Simply take the wheelchair out of its packaging and pull the armrests apart, then push the seat base down until in the horizontal position. The leg rests then clip into the two fixings part way down the wheelchair legs and swing in towards the center of the chair where they will click into place. The leg rests are height adjustable to a number of fixed positions. To adjust the length of the footrest simply push the silver buttons on the footrest pole and slide into one of the fixed height adjustment holes.



## 5. Folding & Transportation

The Global Industrial Transport chair has been designed to fold for ease of transportation and storage. Ensure the footrests are in the forward position and flip the plastic footplates up. Whilst holding the front and rear of the seat canvas, pull upwards to fold the chair.



The backrest also has a half-folding back which can be used to make the chair even more compact. Push the black paddle at the rear of the chair with your thumb whilst pulling the handle down.



Repeat on the paddle on the other side. To return the backrest to the upright position, simply push the two handles back up into the upright position until the backrest locks in place.

**CAUTION:** Do not push down on the handles to raise the front of the wheelchair as this could result in damage to your Transport chair. Use the stepper tube as described further on in this manual.

# 19" Transport Chair

## 6. Transportation

For safe transportation of your transport Chair, first fold the wheelchair as described above. Then remove the leg rests by pushing down on the chrome lever on the leg rest while twisting 90 degrees to the side of the wheelchair. This should unlock the leg rest allowing you to lift the leg rest up and off the wheelchair.

## 7. Stepper Tube and Curbs

Mounting a curb – Approach the curb head on. The attendant pushes down with their foot on the stepper tube at the rear of the wheelchair to raise the front castors off the ground and on to the curb. The attendant then pushes the wheelchair forward and lifts the rear of the wheelchair up onto the curb.



Going down a curb – Approach the curb head on. The attendant pushes down with their foot on the stepper tube at the rear and gently lowers the front castors to the ground. The attendant then pushes the wheelchair forward and gently lowers the rear castors to the ground.

## 8. Safety Seat Belt

The seat belt is designed to restrain the wheelchair occupant during normal use. The lap belt is adjustable in length and should be adjusted to fit the user using the triglides. The lap belt should be tight around the user's pelvis (without causing discomfort) and the user's hips and pelvis should be held in place towards the rear of the Freedom Travel Chair. The seat belt has a luggage style large locking clip to fasten and unfasten. Push the clip into the receiver until it locks into position. The seat belt may not be suitable for all users of the wheelchair. Seek professional medical advice before using the seat belt.

# 19" Transport Chair

## 7. Operation

- The Transport chair can only be propelled by the attendant/carer from the rear of the wheelchair.
- Parking brakes – push the plastic brake handle with your hand towards the front of the wheelchair. Pull the handle back to release the brakes.
- Do not use on gradients larger than 10 degrees.
- The Transport chair is only suitable for a single user.
- Ensure feet are on footplates during operation.
- Do not stand on the footplates.
- Do not use on escalators.
- Do not reverse down a gradient.
- Do not reach for items further than your arm will extend.
- Be aware of hazards in your region when operating.
- Always apply parking brake when transferring into or out of the wheelchair.

## 8. Maintenance

Be sure your Global Industrial Transport chair is in proper working order at all times. Please contact Global Industrial for any repairs required.

- Wipe down the frame using a soft cloth at least once a week.
- If exposed to moisture, dry immediately.
- Polish metal parts of the chair with auto wax once a month.
- Clean the seat upholstery with a mild soap solution, rinse and wipe dry.
- Worn or torn upholstery should be replaced promptly.
- Inspect the wheels once a month—you can correct any side play in the wheel by loosening the lock nut on the axle and then tighten the axle to remove this movement.
- If the Solid PU tire becomes cracked or severely worn, replace immediately