Muñequera ProFlex® (Modelos 4000 y 4010):

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How to put on your ProFlex® Wrist Support (Models 4000/4010):

1. Place around wrist (right or left, as stated) aligning the wrist to the orange line indicator.
2. Fasten hook-and-loop closure on back of wrist.
3. Place palm strap between thumb and index finger. Attach end of strap to hook-and-loop closure on back of wrist.
4. Wrap elastic strap across top of the wrist and fasten at desired tension.

How to put on your ProFlex® Wrist Support (Model 4020):

1. Loosen closure.
2. Slide hand (left or right, as stated) through bottom opening of wrist support. Position palm strap between thumb and index finger.
3. Pull closure tab up and toward the inside of the wrist to meet hook-and-loop closure. Fasten at desired tension.

How to put on your ProFlex® Wrist Support (Models 670/675):

1. Slip hand and thumb through opening.
2. Place removable strap with loop to hook so the strap lays away from the body.
3. Pull strap underneath to wrist side and then over the back of the wrist.
4. Fasten hook-and-loop closure.
5. Repeat Step 2 for the additional strap on Model 675.

Care instructions: Machine wash cold; air dry.

- We cannot guarantee wearing personal protective equipment will prevent injury.

INTERGALACTIC HEADQUARTERS

1021 BANDANA BOULEVARD EAST SUITE 220 SAINT PAUL, MN, USA 55108

©2015 ERGODYNE. ALL RIGHTS RESERVED. ALL WRONGS REVERSED. ERGODYNE, PROFLEX, AND THE ERGOMAN LOGO ARE REGISTERED TRADEMARKS OF ERGODYNE CORPORATION.

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