Health o meter[®]
P r o f e s s i o n a l

Model 450KL Physician Beam Scale

**ISO13485:2003
CERTIFIED**

CE 0459

User Instructions

PELSTAR, LLC 9500 West 55th Street, McCook, IL 60525

Thank you for your purchase of this Health o meter® Professional product. Please read this manual carefully, and keep it for easy reference or training.

TABLE OF CONTENTS

Caution and Warning.....	3
Specifications for This Scale	3
Certifications / Disposal / Service	4
Assembly Instructions 450KL Scale	5
Transportation	10
Operating Instructions	13
Counterweight Instructions.....	16
Height Measurement Instructions.....	23
Troubleshooting	25
Maintenance.....	26
Warranty.....	27

Note: This scale has been factory calibrated, and does not require calibration prior to use.

Please register your scale for warranty coverage at:

www.homscales.com

For User Instructions updates and revisions please go to:

www.homscales.com

CAUTION AND WARNING

- To prevent injury and damage to your scale, please follow these instructions very carefully.
- To prevent injury, as well as scale damage during assembly, exercise caution when assembling the scale pillar.
- Assemble and operate the scale per the enclosed user instructions.
- For accurate weighing, this scale must be placed on a flat, stable surface.
- For accurate weighing, verify before each use the proper operation according to the procedure described in this manual.
- Do not transport the scale with any weight on the platform.
- Do not exceed recommended weight limit of 350 lb / 160 kg or 500 lb / 200 kg with optional counterweights for this scale.
- Do not use in the presence of flammable or explosive materials.
- If the scale becomes damaged, it should not be operated until properly serviced. All repairs should only be performed by authorized Health o meter® Professional service personnel.
- To prevent patient injury, the patient must be attended throughout the entire weighing event.
- Patient should wear socks or other form of lightweight foot covering during weighing event.

SPECIFICATIONS FOR THIS SCALE

General

The Model 450KL physician beam scale employs a proven and rugged mechanical design to measure patient weight and height. Each precision scale is designed to provide accurate, reliable, and repeatable weight measurements. In addition, each scale is designed with a rotating poise bar, and includes counterweights that enable the user to make the weighing process simple, fast, and convenient.

Weight is displayed in pounds (lb) or in kilograms (kg). Height is displayed in inches (in) or in centimeters (cm).

450KL Scale Specifications

Capacity:	350 lb (160 kg)	
Capacity with optional counterweights:	500 lb (200 kg)	
Environmental:	Operating temperatures: 50°F to 104°F (10°C to 40°C) Storage temperatures: 50°F to 104°F (10°C to 40°C) Maximum Humidity: 85% RH	
Physical Dimensions:	Scale (Overall Dimensions) Width: 16-3/4" (42.6 cm) Depth: 20-1/4" (51.4 cm) Height: 57-1/2" (146.1 cm) Weight: 28 lb (12.7 kg)	Platform (Standing Area) Width: 10-1/2" (26.7cm) Depth: 14" (35.6 cm) Height: 3-1/4" (8.3 cm)

CERTIFICATIONS / DISPOSAL / SERVICE

Certification Descriptions

	Pelstar, LLC has been officially certified as the manufacturer of Health o meter [®] Professional medical devices. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.
	Products carrying this symbol meet the requirements of: 1. Directive governing medical devices 93/42/EEC

450KL Scale Disposal

This Health o meter[®] Professional scale must be disposed of properly as steel waste. Follow the national, regional or local regulations which apply to you for disposal of steel waste.

CUSTOMER SERVICE INFORMATION

The 450KL physician beam scale is shipped disassembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact Customer Service.

For further information or telephone support, please contact Customer Service at:

Telephone: 1 (800) 815-6615

Email: homprocs@homscales.com

450KL ASSEMBLY INSTRUCTIONS

Before Assembly

Each 450KL physician beam scale is shipped disassembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact your shipper or a Health o meter® Professional representative immediately. Claims must be filed with the shipper as soon as possible after receipt of the package. The following information details what you will find inside the main carton as you unpack the parts for assembly.

To prevent scratching any components and avoid damaging the scale parts when unpacking, do not use a box cutter, knife, scissors, or any sharp object to open the protective inner packaging. Carefully remove each assembly from the carton and unwrap the packing materials. Set the carton aside for storage.

Parts List

Carton

- (1) Scale platform
- (1) Pillar with height rod attached
- (4) Pillar nuts
- (1) User instructions
- (1) Wrench
- (1) Counterweight set (Part Number 55070)
 - 150 lb counterweight (Part Number 55060)
 - 40 kg counterweight (Part Number 55065)

Tools Required

Wrench (included)

450KL ASSEMBLY INSTRUCTIONS (CONTINUED)

Assembling the 450KL Scale (Refer to Figures 1-6 on pages 7-9)

Caution: It is recommended that the assembly be performed by two persons.

1. Carefully remove the scale platform and pillar from the carton. Place the scale platform and pillar on a flat, level, and dry surface. Using a table can be helpful for assembly of 450KL scale (see Figures 1 & 5 on pages 7 & 9).
2. Remove the four pillar nuts and wrench from the carton.
3. Locate the steel rod at the base of the pillar (see Figures 4, 5 & 6 on pages 8 & 9).
4. Cut the tie wrap that holds the steel rod to the pillar studs (see Figure 3 on page 8).
5. Insert studs into the four holes provided in the scale platform (see Figure 2 on page 7).
6. Tip the scale platform at an angle, resting the top of the pillar on any convenient support. Using the wrench provided, secure the pillar to the scale platform with the four nuts (see Figures 4 & 5 on pages 8 & 9).
7. Pull the steel rod toward the base of the scale platform by grasping the finger pull loop on the steel rod end (see Figures 4, 5 & 6 on pages 8 & 9).
8. Push the lever extension toward the pillar with your hand (see Figure 4, 5 & 6 on pages 8 & 9).
9. While holding this position, hook the steel rod to the hole at the end of the lever extension. Release the finger pull loop and ensure that the steel rod and lever extension is hooked together.
10. Carefully position the scale to its upright position.
11. The scale is now assembled and is ready for use (see Figure 1 on page 7).

Note: To transport your scale, acquire the optional wheel bracket (Part Number # 55000). Please refer to the "Transportation" section for assembly instruction.

450KL ASSEMBLY INSTRUCTIONS (CONTINUED)

Figure 1

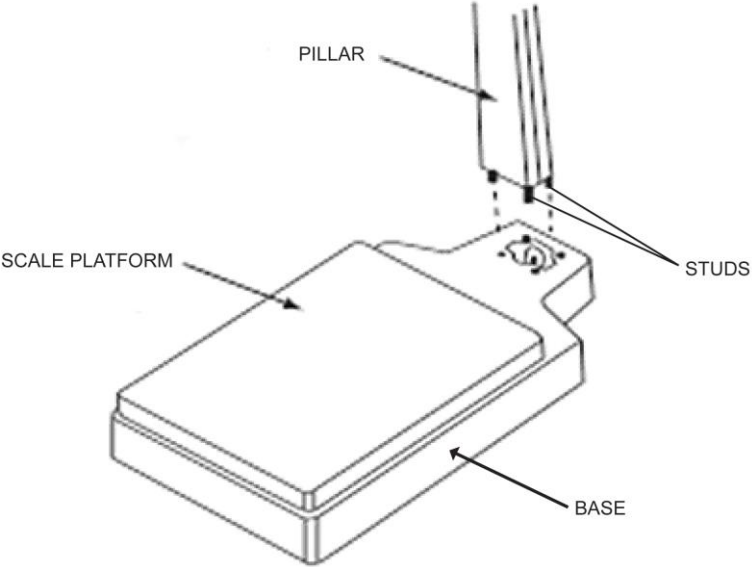
Figure 2

450KL ASSEMBLY INSTRUCTIONS (CONTINUED)

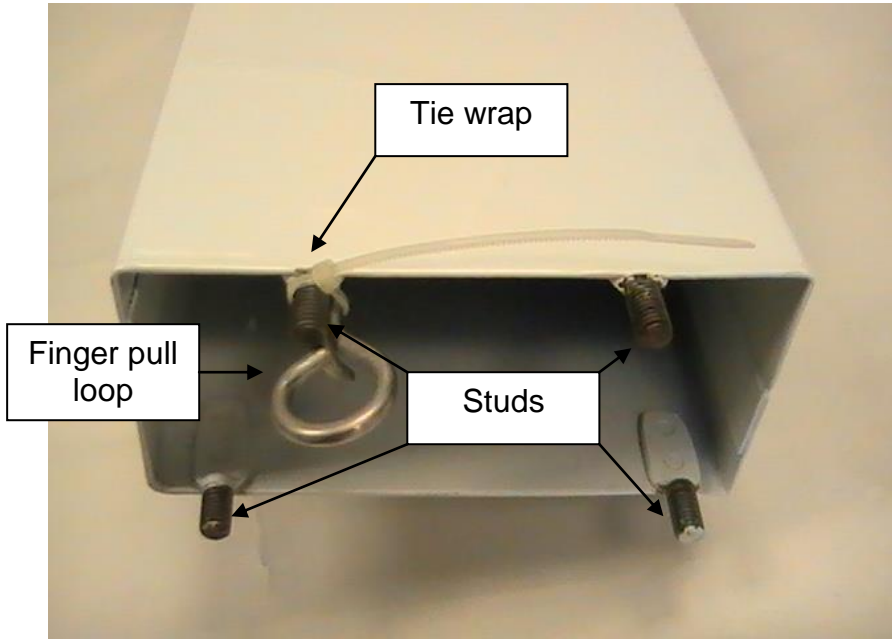
Figure 3

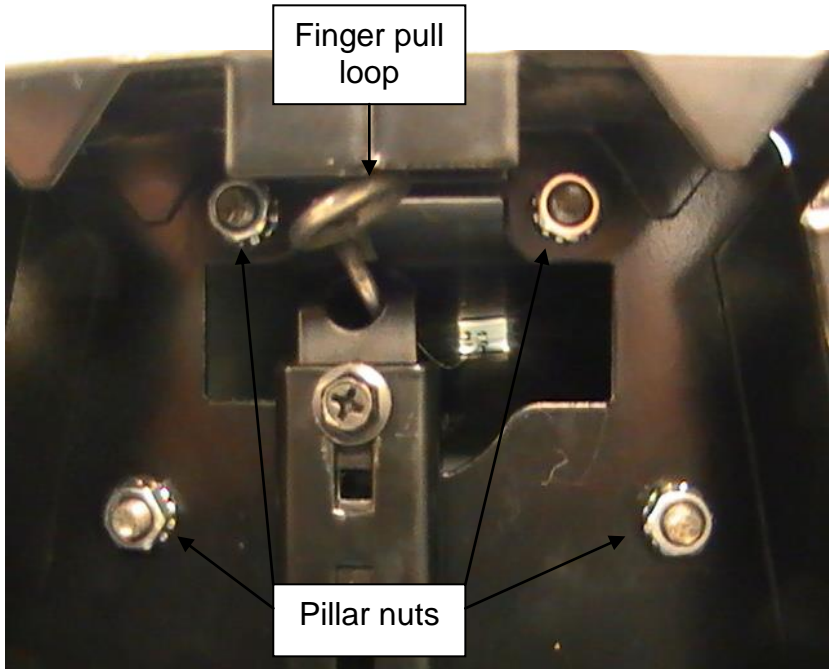
Figure 4

450KL ASSEMBLY INSTRUCTIONS (CONTINUED)

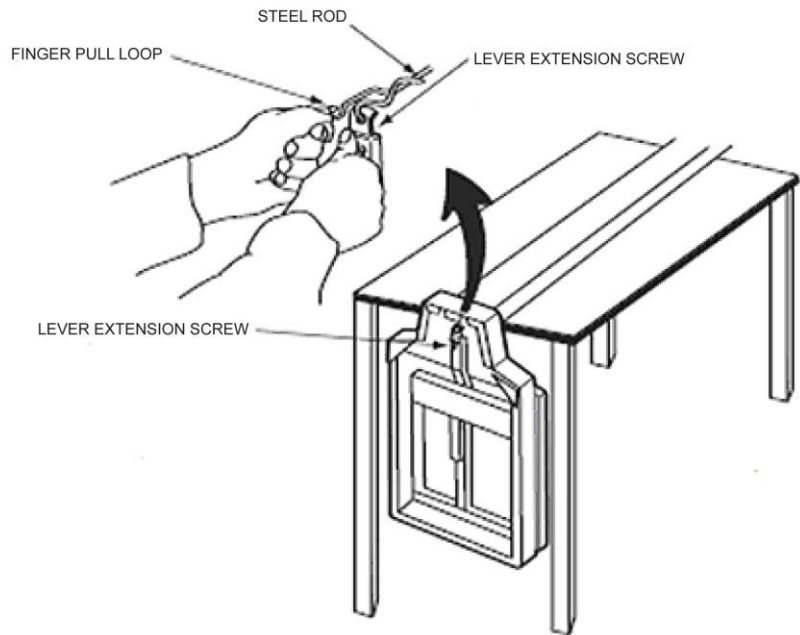
Figure 5

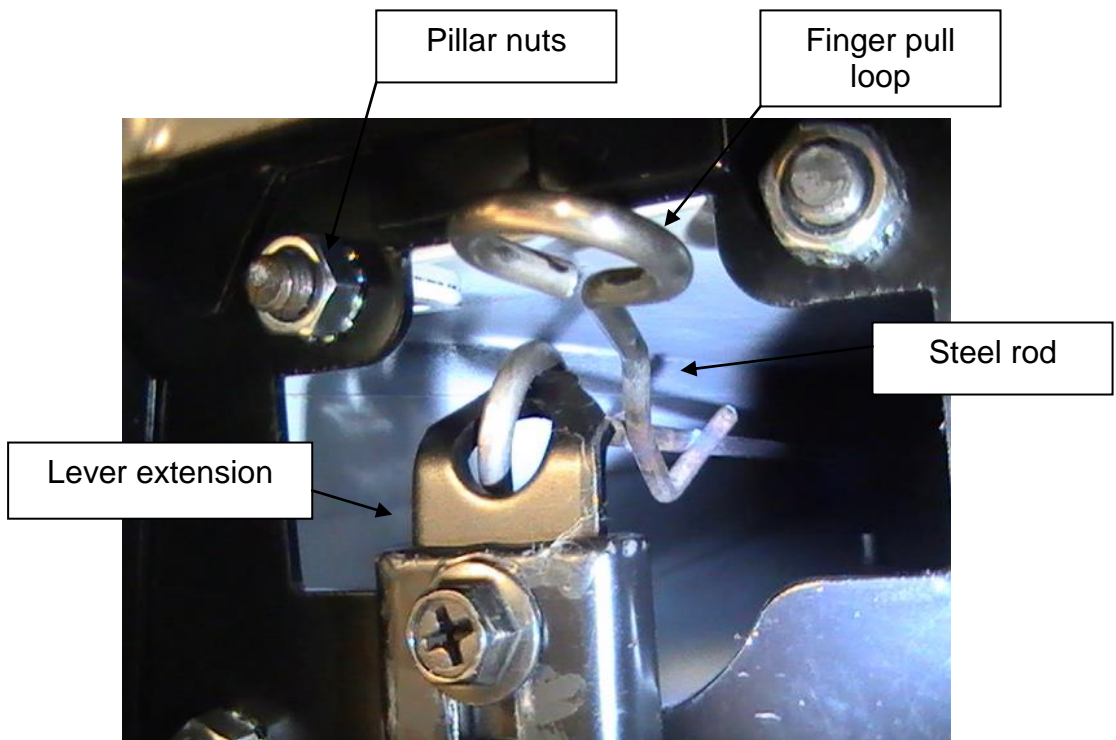
Figure 6

TRANSPORTATION

Preparing a 450KL Scale for Transportation (see Figures 7-10 on pages 11 & 12)

This scale can be easily assembled with a wheel bracket (Part Number 55000, see Figure 7 on page 11) for storage, or for transport from one room to another. This scale does not include 55000 Wheel Bracket, and must be acquired separately. To prepare the scale for transport, follow these steps:

Caution: It is recommended that the assembly be performed by two persons.

1. Remove the mounting screws attaching the clamping plate to the wheel bracket. (see Figure 8 on page 11)
2. Tip the scale forward until top front of the pillar is resting on the floor.
3. Position the clamping plate inside the base, and align the clamping plate, wheel bracket, and base assembly holes. (see Figure 9 on page 12)
4. Install the mounting screws through the wheel bracket, base, and clamping plate. Tighten the mounting screws securely. (see Figure 10 on page 12)
5. Carefully tip the scale to its upright position.
6. To transport the scale, tip the scale pillar back until the scale is supported by the wheel bracket. Carefully move the scale to its new location.

Required Parts

- (1) 450KL
- (1) Wheel Bracket (Part Number 55000)

Note: Wheel bracket is not included.

Tools Required (not included)

Adjustable wrench or 10mm wrench

TRANSPORTATION (CONTINUED)

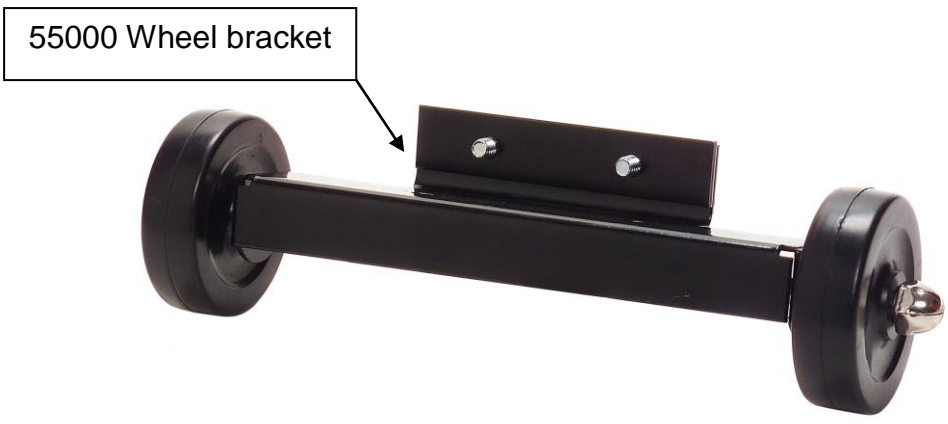
Figure 7


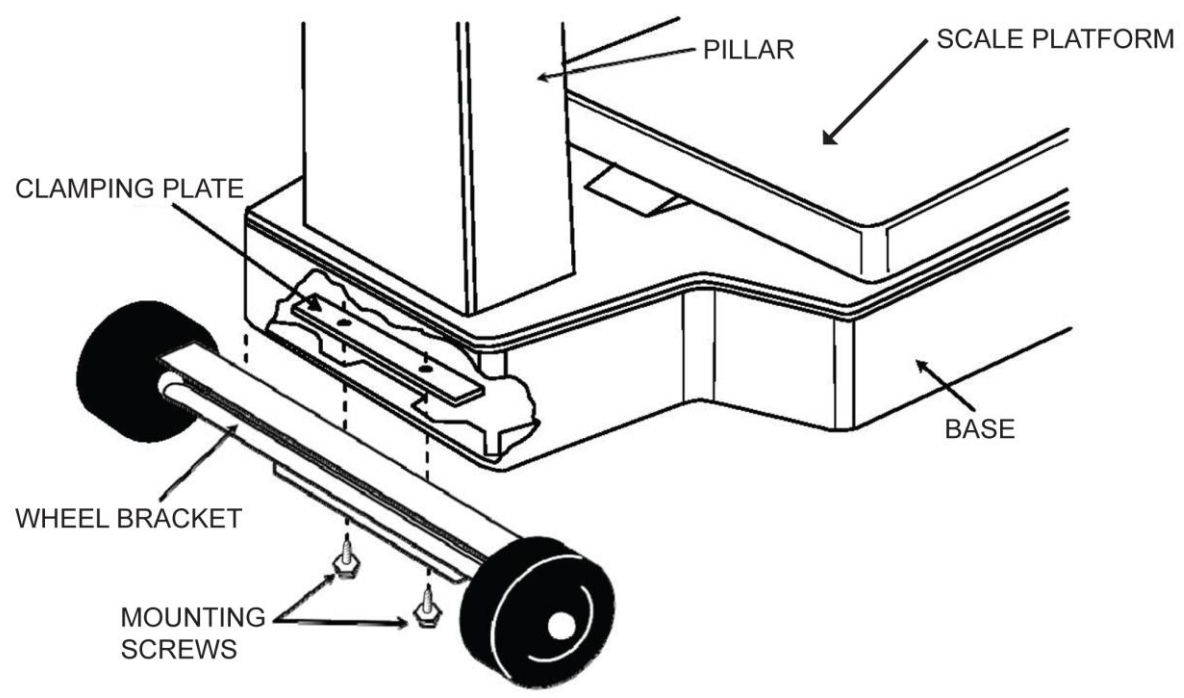
Figure 8


TRANSPORTATION (CONTINUED)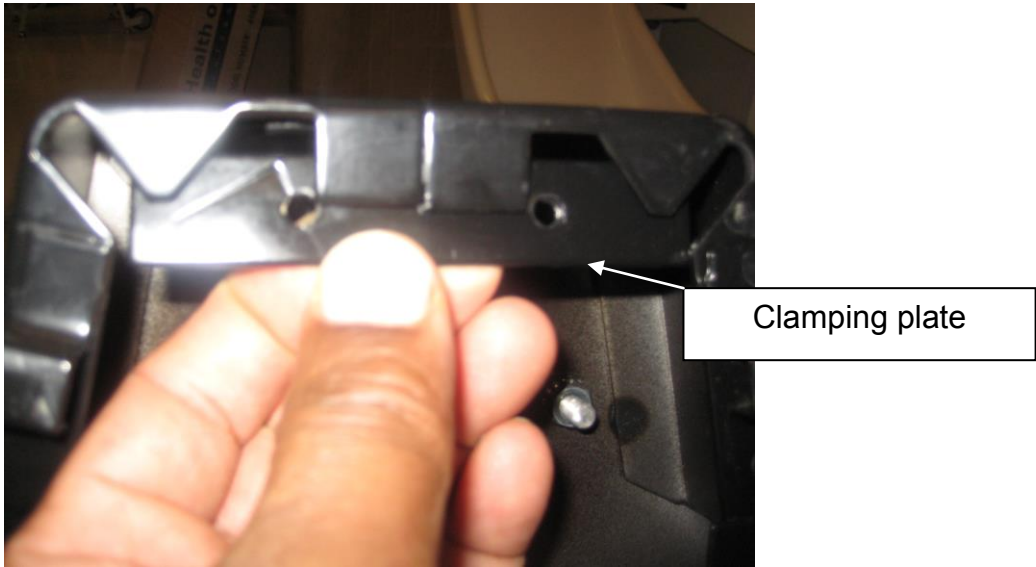

Clamping plate

Figure 9

Wheel bracket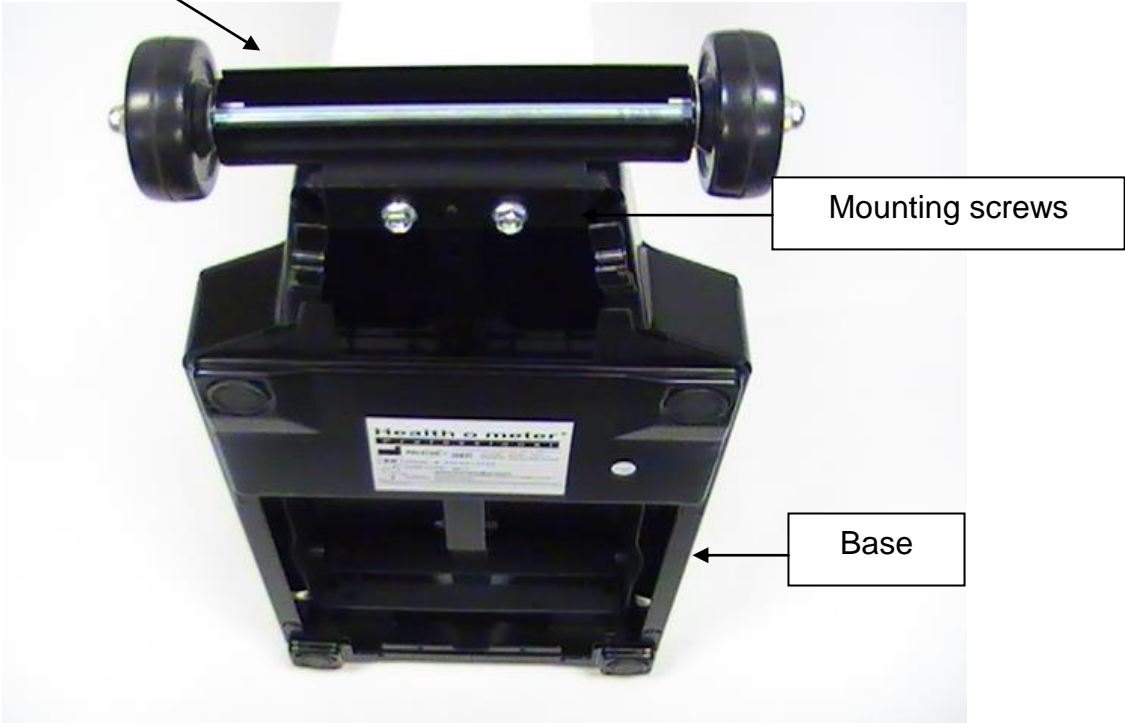

Mounting screws

Base

Figure 10

OPERATING INSTRUCTIONS

Zero Balance Adjustment

The scale should be checked periodically, particularly when moved, by following the instructions below:

Note: A flat head screwdriver is required.

1. Ensure that the scale is located on a hard, flat, and level surface. Step on and off scale several times in order to align all internal parts.
2. Place both poise weights on the “0” value on both bars. The beam pointer should float gently up and down, and not touch the top or bottom of the trig-loop.
3. If the beam pointer is not centered in the trig-loop opening, rotate balance ball screw to correct the beam pointer.

Weighing Instructions (see Figures 11–14 on pages 14 & 15)

1. Set the large and small poises to indicate the patient’s approximate weight.
2. Have patient step onto the scale. Ensure they are positioned directly in the center of the platform. Continue to adjust the small and large poise weights until the pointer is centered.

Note: It is not necessary for you to wait for the pointer to stop moving in order to read the correct weight. As long as the pointer is moving equally within the trig-loop opening, the patient’s weight can be determined.

3. The patient weight can be recorded in either pounds or kilograms.
4. Observe the location of the indicator arrow built into each poise weight. Read the value on the poise bar, closest to the indicator arrow.
5. Rotate the upper and lower poise bars to measure in either pounds or kilograms. Ensure that poise bars are locked after being rotated into position.
6. For easy identification, pounds (lb) have silver graduations on a black background, located on one side of the poise bar (see Figure 11 on page 14); kilograms (kg) have black graduations on a silver background, located on the other side of the poise bar.(see Figure 12 on page 14)
7. Add the indicated reading of the small poise value to the indicated reading of the large poise value. The total amount is the patient’s weight. (see Figures 13 and 14 on page 15)

OPERATING INSTRUCTIONS (CONT)

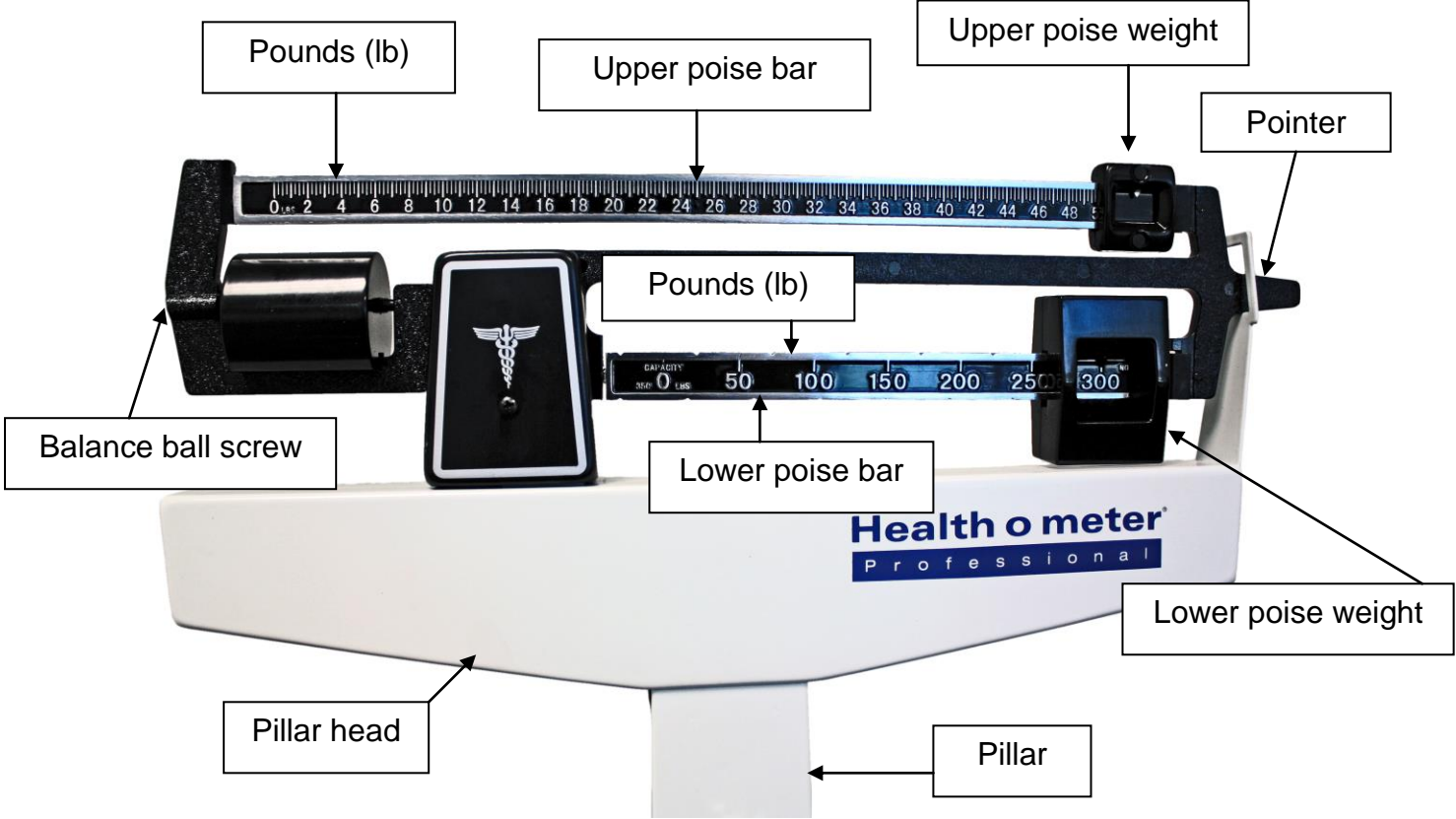
Figure 11

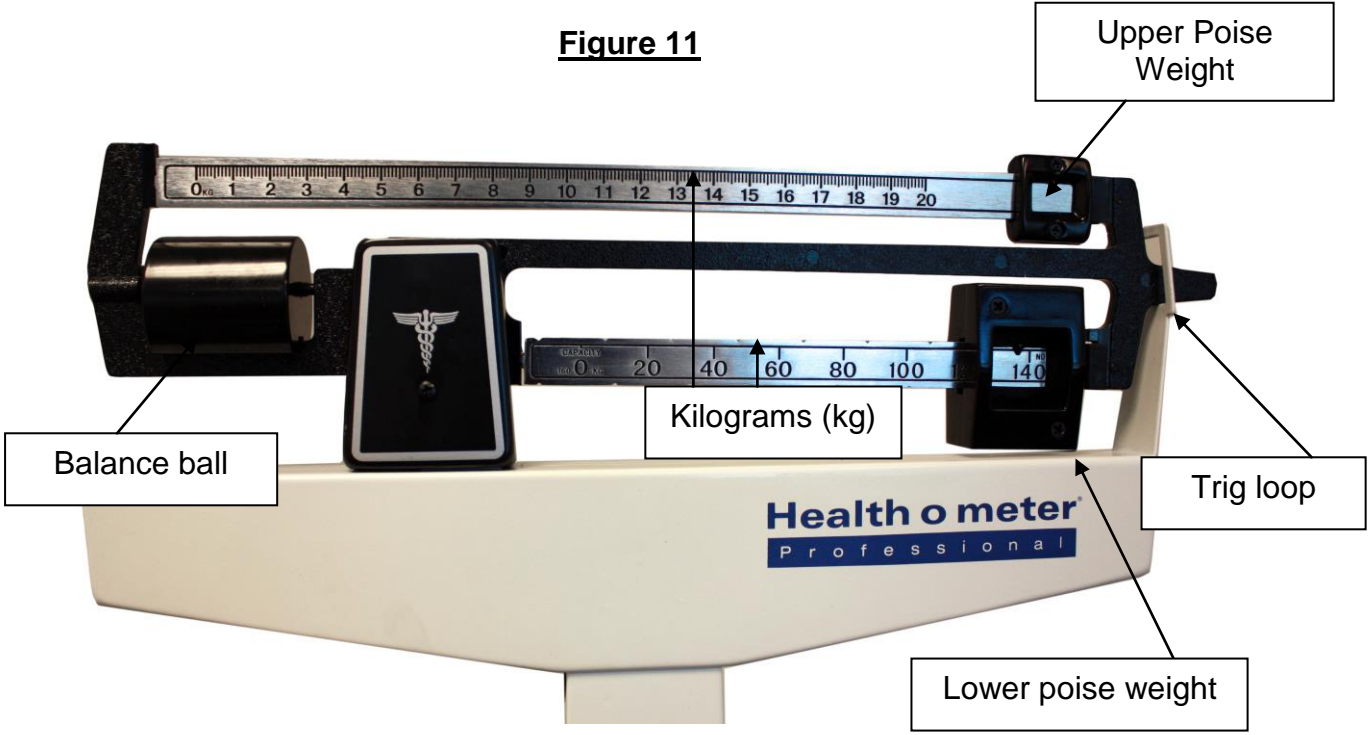
Figure 12

OPERATING INSTRUCTIONS (CONT)

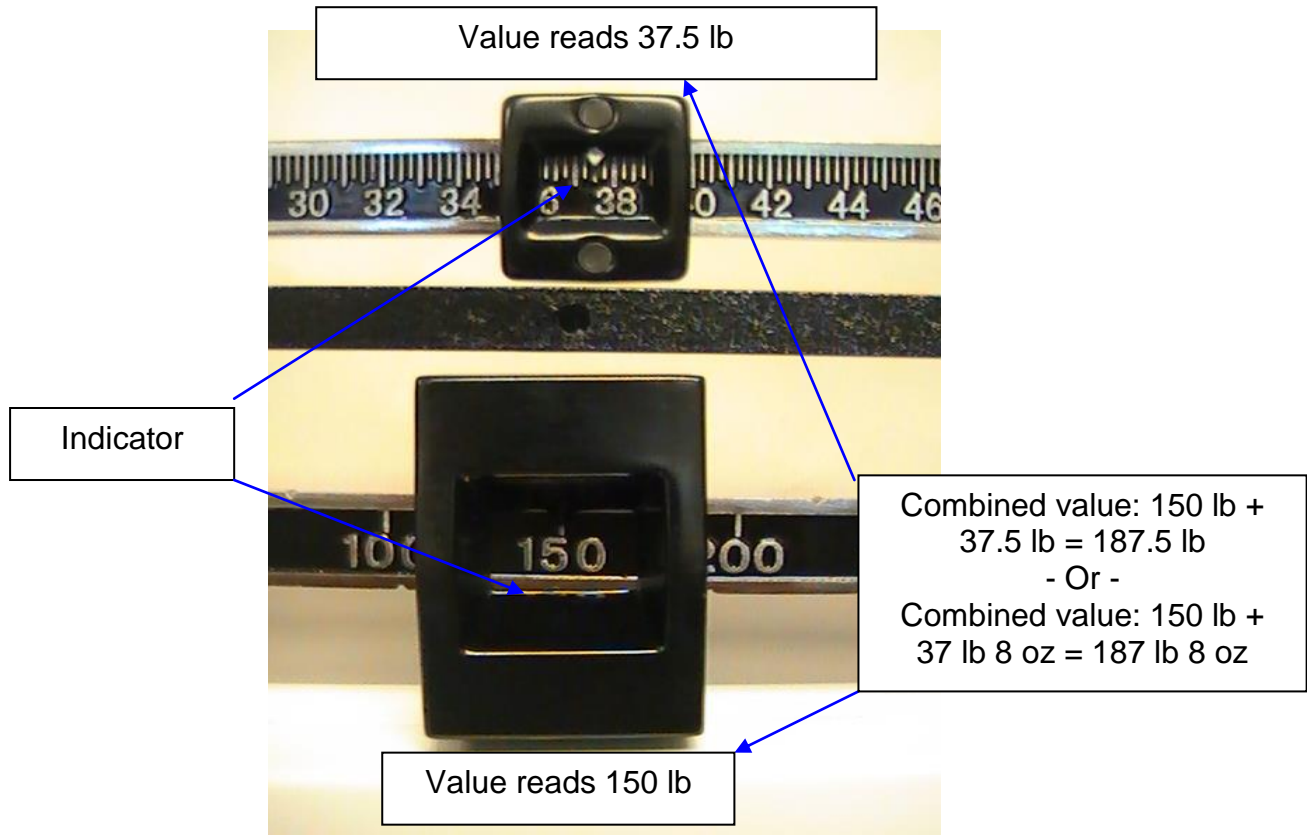
Figure 13

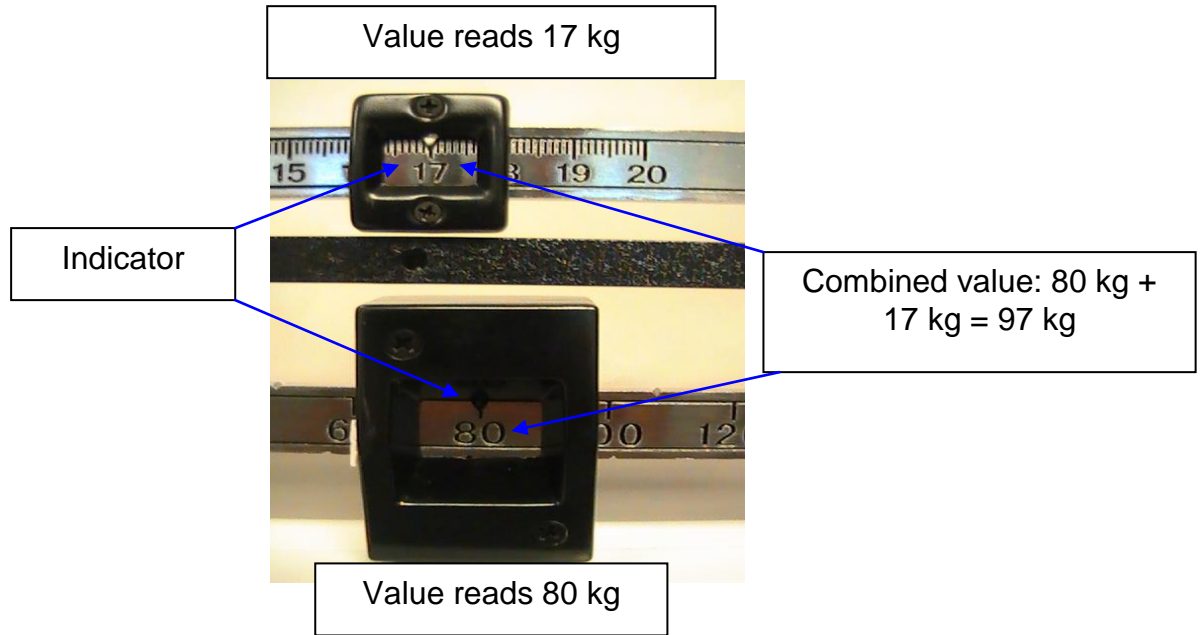
Figure 14

COUNTERWEIGHT INSTRUCTIONS

Counterweight Instructions (see Figures 15-23 on pages 19-22)

In order to increase the scale's capacity to a maximum value of 500 lb / 200 kg, the counterweight set (55070, see Figure 15 on page 19) must be used.

In order to employ the counterweight set, refer to the instructions below. Contact technical support with any questions or concerns regarding the use of the counterweight set.

Counterweight Instructions for Weighing in Pounds (lb)

Note: The 150 lb counterweight is required when weighing a patient who is over 350 lbs.

Counterweight Instructions for Weighing a Patient 351 - 388 (lb) (see Figures 16 & 18 on pages 19 & 20)

1. When a patient weighs between 351 - 388 lb, place the lower poise weight at 250 lb.
2. Place the 150 lb counterweight on the 3rd notch from the left, between the 50 and 100 lb value.
3. Adjust the small poise weight to match the patient weight. The poise bar will center in the trig-loop.

Note: To read a patient weight refer to: "Recording a Patient Weight" section below.

4. Read and record the weight.

Counterweight Instructions for Weighing a Patient 389 - 424 (lb) (see Figures 16 & 19 on pages 19 & 21)

1. When a patient weighs between 389 - 424 lb, place the lower poise weight at 300 lb.
2. Place the 150 lb counterweight on the 3rd notch from the left, between the 50 and 100 lb value.
3. Adjust the small poise weight to match the patient weight. The poise bar will center in the trig-loop.

Note: To read a patient weight refer to, "Recording a Patient Weight" section below.

4. Read and record the weight.

Counterweight Instructions for Weighing a Patient 425 - 450 (lb) (see Figures 16 & 20 on pages 19 & 21)

1. When a patient weighs between 425 - 450 lb, place the lower poise weight at 250 lb.
2. Place the 150 lb counterweight on the 5th notch from the left, between the 150 and 200 lb value.
3. Adjust the small poise weight to match the patient weight. The poise bar will center in the trig-loop.

Note: To read a patient weight refer to, "Recording a Patient Weight" section below.

4. Read and record the weight.

COUNTERWEIGHT INSTRUCTIONS (CONT)

Counterweight Instructions for Weighing a Patient 451 - 500 (lb) (see Figures 16 & 21 on pages 19 & 21)

1. When a patient weighs between 451 - 500 lb, place the lower poise weight at 300 lb.
2. Place the 150 lb counterweight on the 5th notch from the left, between the 150 and 200 lb value.
3. Adjust the small poise weight to match the patient weight. The poise bar will center in the trig-loop.

Note: To read a patient weight refer to, "Recording a Patient Weight" section below.

4. Read and record the weight.

Counterweight Instructions for Weighing in Kilograms (kg)

Note: The 40 kg counterweight is required when weighing a patient who is over 160 kg.

Counterweight Instructions for Weighing a Patient 161 - 180 (kg) (see Figures 17 & 22 on pages 20 & 22)

1. When a patient weighs between 161 - 180 kg, place the lower poise weight at 120 kg.
2. Place the 40 kg counterweight on the 5th notch from the left, between 60 and 80 kg value on the lower poise bar.
3. Adjust the small poise weight to match the patient weight. The poise bar will center in the trig-loop.

Note: To read a patient weight refer to: "Recording a Patient Weight" section below.

4. Read and record the weight.

Counterweight Instructions for Weighing a Patient 181 - 200 (kg) (see Figures 17 & 23 on pages 20 & 22)

1. When a patient weighs between 181 - 200 kg, place the lower poise weight at 140 kg.
2. Place the 40 kg counterweight on the 5th notch from the left, between 60 and 80 kg value on the lower poise bar.
3. Adjust the small poise weight to match the patient weight. The poise bar will center in the trig-loop.

Note: To read a patient weight refer to: "Recording a Patient Weight" section below.

4. Read and record the weight.

Recording a Patient Weight in Pounds (lb) between 351 - 424 lb

To read the patient's weight that is between 351 lb - 424 lb, add the weight value indicated on the lower and upper poise bars, and add 88 lb.

For example: If a patient weighs 412 lb:

1. When a patient weighs 412 lb, place the lower poise weight at 300 lb.
2. Place the 150 lb counterweight on the 3rd notch from the left, between the 50 and 100 lb value.
3. Adjust the small poise weight to match the patient weight. The poise bar will center in the trig-loop.
4. Add the weight value indicated on the upper and lower poise bars, as well as an additional 88 lb:

$$300 \text{ lb} + 24 \text{ lb} + 88 \text{ lb} = 412 \text{ lb}$$

Patient's weight is: 412 lb

COUNTERWEIGHT INSTRUCTIONS (CONT)

Recording a Patient Weight in Pounds (lb) between 425 - 500 lb

To read the patient's weight that is between 425 lb - 500 lb, add the weight value indicated on the lower and upper poise bars, and add 150 lb.

For example: If a patient weighs 441 lb:

1. When a patient weighs 441 lb, place the lower poise weight at 250 lb.
2. Place the 150 lb counterweight on the 5th notch from the left, between the 150 and 200 lb value.
3. Adjust the small poise weight to match the patient weight. The poise bar will center in the trig-loop.
4. Add the weight value indicated on the upper and lower poise bars, as well as an additional 150 lb:

$$250 \text{ lb} + 41 \text{ lb} + 150 \text{ lb} = 441 \text{ lb}$$

Patient's weight is: 441 lb

Recording a Patient Weight in Kilograms (kg) between 161 - 180 kg

To read the patient's weight that is between 161 – 180 kg, add the weight value indicated on the lower and upper poise bars, and add 40 kg.

For example: If a patient weighs 171 kg:

1. When a patient weighs 171 kg, place the lower poise weight at 120 kg.
2. Place the 40 kg counterweight on the 5th notch from the left, between the 60 and 80 kg value.
3. Adjust the small poise weight to match the patient weight. The poise bar will center in the trig-loop.
4. Add the weight value indicated on the upper and lower poise bars, as well as an additional 40 kg:

$$120 \text{ kg} + 11 \text{ kg} + 40 \text{ kg} = 171 \text{ kg}$$

Patient's weight is: 171 kg

Recording a Patient Weight in Kilograms (kg) between 181 - 200 kg

To read the patient's weight between 181 – 200 kg, add the weight value indicated on the lower and upper poise bars, and add 40 kg.

For example: If a patient weighs 192 kg:

1. When a patient weighs 192 kg, place the lower poise weight at 140 kg.
2. Place the 40 kg counterweight on the 5th notch from the left, between the 60 and 80 kg value.
3. Adjust the small poise weight to match the patient weight. The poise bar will center in the trig-loop.
4. Add the weight value indicated on the upper and lower poise bars, as well as an additional 40 kg:

$$140 \text{ kg} + 12 \text{ kg} + 40 \text{ kg} = 192 \text{ kg}$$

Patient's weight is: 192 kg

COUNTERWEIGHT INSTRUCTIONS (CONT)

Counterweight Set (55070)

Figure 15

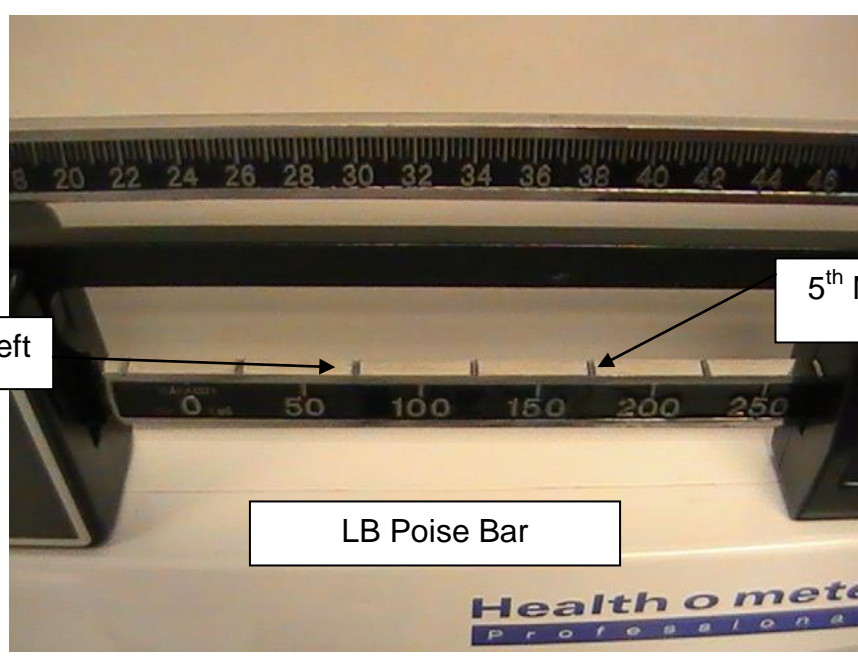
Figure 16

COUNTERWEIGHT INSTRUCTIONS (CONT)

5th Notch from the left

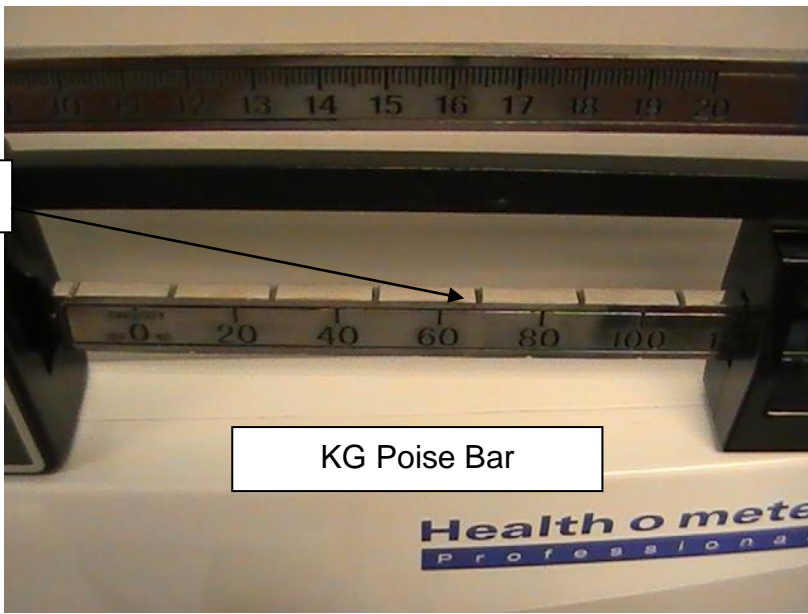
Figure 17

351 - 388 lb Assembly

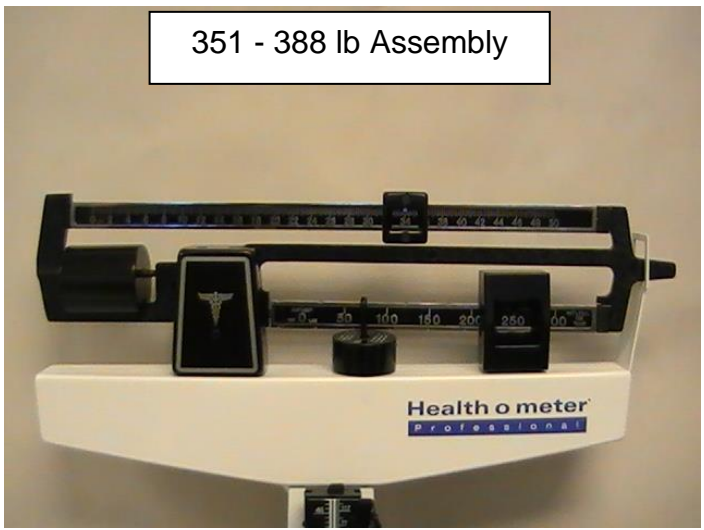
Figure 18

COUNTERWEIGHT INSTRUCTIONS (CONT)

Figure 19

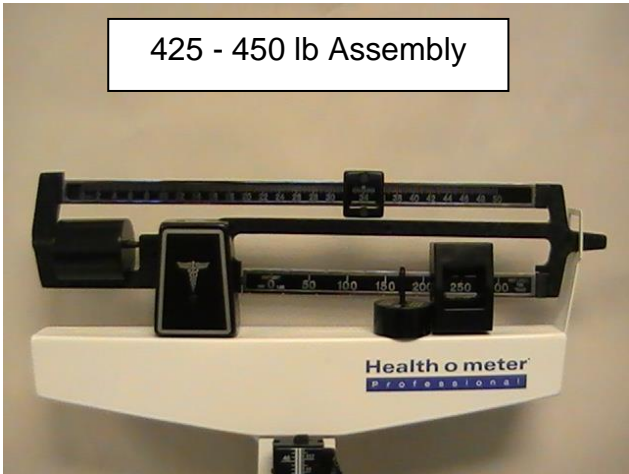
Figure 20

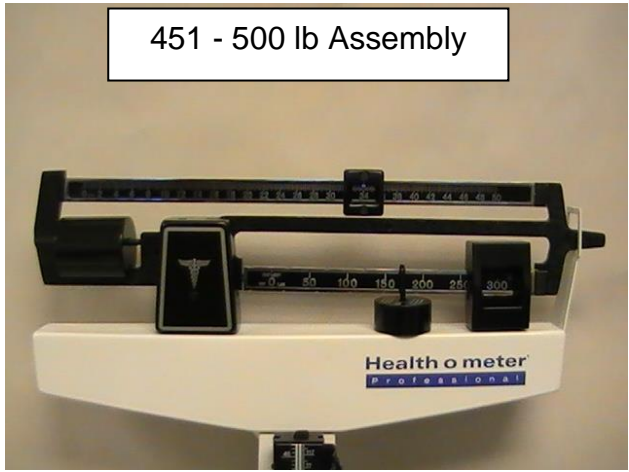
Figure 21

COUNTERWEIGHT INSTRUCTIONS (CONT)

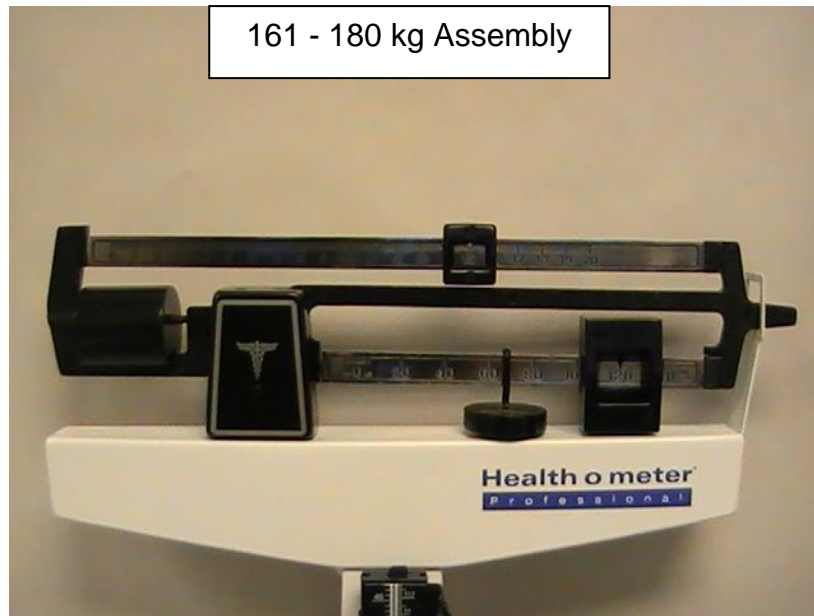
Figure 22

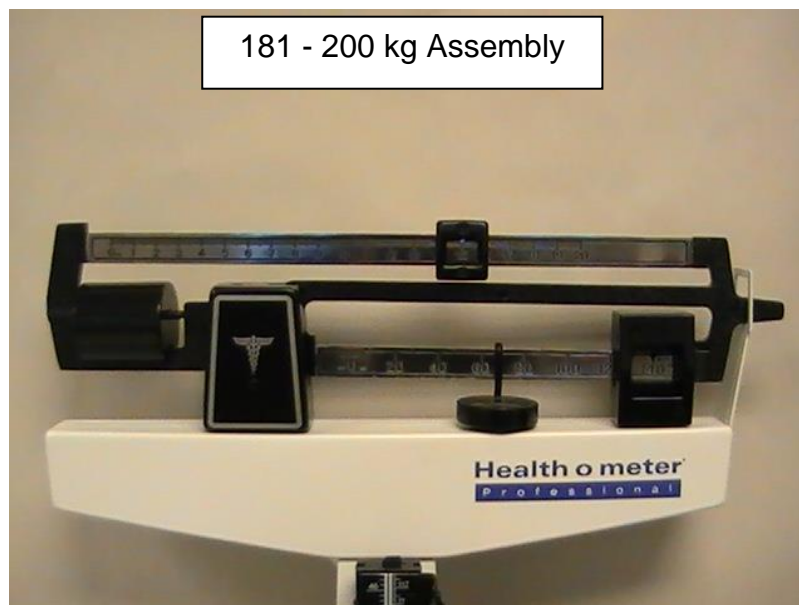
Figure 23

HEIGHT MEASUREMENT INSTRUCTIONS

Note: Use caution when sliding height rod up and down. The rod should move smoothly, do not move the rod in a jerking motion.

Note: Fold headpiece down when not in use.

Height Measurements (Refer to figures 24-30)

Note: A patient needs to be a minimum of 23-5/8" (600mm) in height for the height to be accurately measured.

1. Raise the headpiece above the patient's head. (see Figures 24 and 26)
2. Position the patient against the height rod.
3. Ensure that the patient's posture is upright, and that the patient's head is straight and level.
4. Push the headpiece down until it rests on the patient's head. (see Figures 25 and 27)
5. Determine the patient's height by reading the value on the measuring strip nearest to the line on the indicator (see Figures 28 – 30 on page 24).

Figure 24

Figure 25

Figure 26


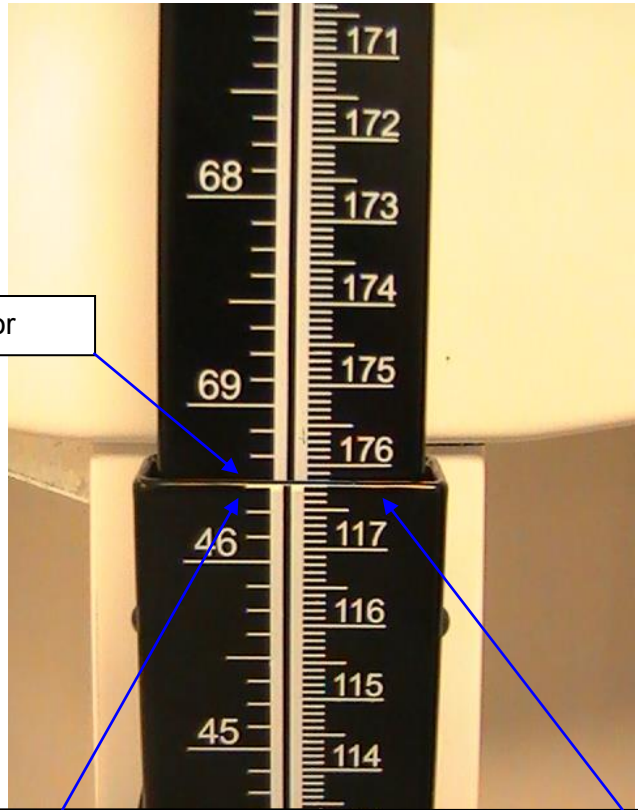
Figure 27

HEIGHT MEASUREMENT INSTRUCTIONS

Height Rod Below 46 3/8"

Height Rod Extended Above 46 3/8"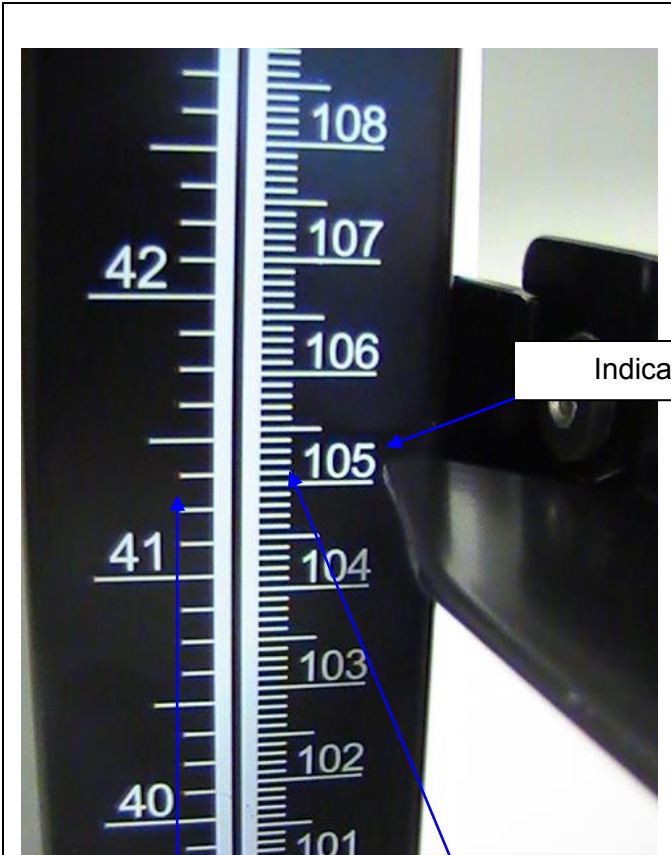

Indicator

Inches read: 41" + 3/8" = 41 3/8"


Cm read: 105 cm + 0.2 cm = 105.2 cm

Inches read: 69" + 3/8" = 69 3/8"

Cm read: 176 cm + 0.2 cm = 176.2 cm

Figure 28

Figure 29

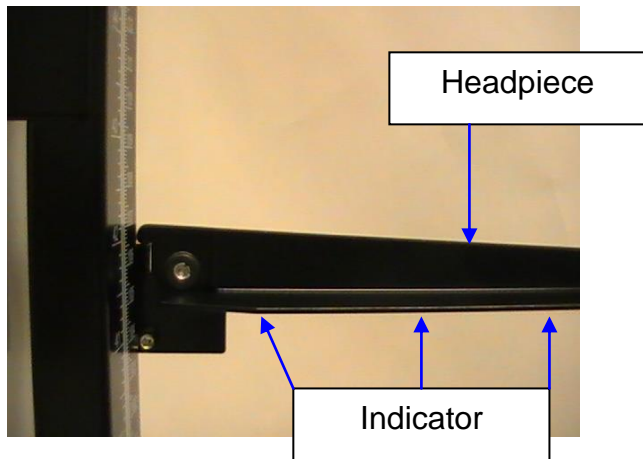Headpiece

Indicator

Figure 30

TROUBLESHOOTING

Troubleshooting

Before contacting service personnel, refer to the following instructions to check and to correct any failures for incorrect weight measurements.

The accuracy of the 450KL scale when calibrated at the factory is within +/- 0.1kg or ¼ pound. For the most accurate weight measurement, use the scale on a hard level surface and stand in the center of the platform. If the error seems excessive, check the following:

Symptom	Possible Cause	Corrective Action
Questionable weight, or the scale does not center equally in the trig loop when the poise weight is "0" value	Bar does not "0" balance	The balance ball must be balanced such that the pointer comes to rest in the center of the trig-loop when both poise weights are set at "0" value. Ensure that the large and small poise weights are firmly against the shoulder of the beam. Adjust the balance ball screw by turning the screw at the left end of the bar until balanced.
Bar does not move freely	Bar sticking or adjustment needed	Check if the pointer is touching the side of the trig loop in its range of travel. Realignment of the beam may be necessary if it is touching. Ensure that the large and small poise weights are firmly against the shoulder of the beam. Adjust the balance ball screw by turning the screw at the left end of the bar.
Platform rocks excessively or touches base at any corner	Base out of alignment	Contact technical support at 1(800)-638-3722.
Beam does not move at all during weighing process	Scale may have improper steel rod connection or poise weight may have been set at a higher weight than patient's actual weight.	Reset the weights to a lower weight. Ensure the steel rod is connected properly, refer to "Assembly Instructions" on page 5.
Scale not accurately measuring weight	Scale out of calibration.	The scale can be recalibrated by loosening the lock screw at the end of the long lever in the bottom of the base with a ¾" hexagon socket wrench and moving it outward (longer) in the elongated slot to decrease the indicated weight or inward (shorter) to increase the indicated weight. One sixteenth inch movement of the screw will change the indicated weight by approximately one-quarter pound per 100 pounds.

For further assistance please contact technical support toll free at 1 (800) 638-3722.

MAINTENANCE

The following provide instructions for maintenance and cleaning for the 450KL. Maintenance operations other than those described in this manual should be performed by qualified service personnel.

MAINTENANCE

Before first use and after periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.

1. Check overall appearance of the total scale for any obvious damage, wear and tear.
2. Do not store near chemicals, heaters or radiators.
3. Do not drop, kick, or jump on the scale.
4. This scale is a sensitive weighing device. Items such as clothing or towels placed or dropped on the scale may cause it to operate incorrectly. In order to prevent this, do not store or leave anything on the scale.
5. If your scale fails to work properly, do not attempt to dismantle it. This will void your warranty. See warranty information instructions on having your scale serviced.
6. All bearings and working parts are permanently lubricated. **DO NOT USE OIL OR LUBRICANTS.**

CLEANING

Proper care and cleaning is essential to ensure a long life of accurate and effective operation.

1. Clean all external surfaces with a clean damp cloth or tissue. Mild antimicrobial or antibacterial soap and water solution may be used. Dry with a clean soft cloth.
2. Do not immerse the scale into cleaning or other liquid solution.
3. Do not use abrasive cleaners.

Replacement Parts

PART NUMBER	DESCRIPTION	UNIT OF MEASURE
400HARDWARE	ALLEN WRENCH AND HEX SCREWS	1 SET
55000	WHEELS	1 EA
55070	COUNTERWEIGHTS	2 EA
55060	LB COUNTERWEIGHT	1 EA
55065	KG COUNTERWEIGHT	1 EA
402KLROD	HEIGHT ROD	1 EA
400BASE	BASE	1 EA
400COVER	PLATFORM COVER	1 EA

WARRANTY

10-Year Limited Warranty

What does the Warranty Cover?

Health o meter® Professional scales are warranted from date of purchase against defects of materials or in workmanship for a period of ten (10) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC
Attention R/A# _____
Return Department
9500 West 55th Street
McCook, IL 60525

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PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:

www.homscales.com

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license. Health o meter® Professional products are manufactured, designed, and owned by Pelstar, LLC. We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

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© Pelstar, LLC 2018
Health o meter® est une marque de commerce enregistré de Sunbeam Products, Inc. Utilisée sous licence.
Les produits Health o meter® Professional sont manufacturés, conçus, et possédés par Pelstar, LLC.