# Two-Gallon Sharps Container

Model 102 / Modelo 102 / Modèle 102 (7,6 L)

*See instructions  
for use on back.*

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## **ATTENTION**

**Please review these instructions carefully  
before using this container.**

- For use in supervised and controlled areas
- Please check that container has not been damaged before placing into use.
- Never hold sharps container below recommended fill line.
- Examine autoclaved parts before handling.
- If container is dropped, inspect container for damage before handling.
- Do not fill sharps container above recommended fill line.
- For single use only. (2)
- Not made with natural rubber latex.
- Designed to comply with heavy metal limits as listed in RoHS/Directive on the Restriction of the use of Certain Hazardous Substances in Electrical and Electronic Equipment (2002/95/EC).
- Safe for incineration.

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## 1. To assemble

- Align front of lid (lid opening) to front of base (biohazard label).
- Align tabs to slots in base. If they do not align, make sure lid and base are both facing forward.
- Press lid to base until all tabs snap into slots and are secure.

## 2. To mount in bracket

- Mount on wall using wall mounting bracket #445 020. Gently slide container into bracket until it clicks.

## 3. To use

- With one hand, drop syringe horizontally into opening.
- Check fill level regularly. Replace container when fill level is reached.

## 4. Temporary closure

Lid can be temporarily closed when not in use. With tab facing up, bring lid forward and press down on two front snaps. Use tab to reopen lid.

## 5. Final closure

- When full, bring lid forward and insert tab into slot. Press down on two front snaps to lock lid closed.
- To remove container from bracket, insert the bracket key into slots at bottom of unit, push up and release container.

## 6. Disposal

Dispose of container according to facility policy. May be autoclaved or incinerated.

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